UltraCal® XS

Calcium Hydroxide Paste

EN

Description:
UltraCal® XS is an aqueous, 12.5 pH, syringeable, radiopaque, calcium hydroxide paste. Use it as an endodontic treatment dressing. Superior delivery control of UltraCal XS is obtained with a NaviTip™ attached.

Indications:
UltraCal XS is to be used as per literature teachings for pulp capping, lining pulpotomies, or as a temporary dressing as per apexification procedures in endodontics.

Procedure:
Following instrumentation, remove luer lock cap from UltraCal XS syringe and attach a NaviTip. Deliver UltraCal XS short of the apex while slowly withdrawing tip (Fig. 1).

Note: Use Ultra-Blend® plus™ for pulp capping procedure.

Precautions:
1. Material is basic. Use carefully and rinse with lots of water if material is inadvertently discharged in an undesired area.
2. Tips are disposable.
3. Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
4. Deliver with NaviTip, staying at least 2mm shy of the apex, always keeping the delivery tip loose in the canal. Don't deliver with plastic tips which can occlude tight in the canal (example Capillary™ tips).
5. Prefilled syringes are multi-dosage. Clean and disinfect between applications as appropriate.
6. We highly recommend using Ultradent® Syringe Covers sealed with the Impulse Sealer to provide a protective barrier against cross-contamination.

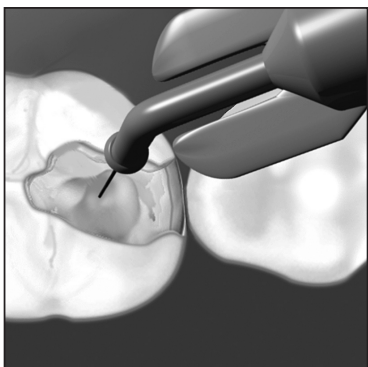
Fig. 1 Following initial instrumentation, canal is filled with UltraCal XS paste.

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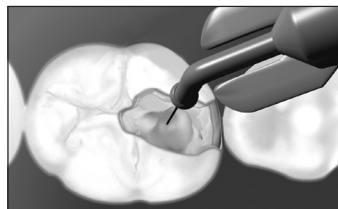
EN Following instrumentation, fill canal from apical portion of root outward. Stay at least 2mm shy of apex.

Canal prior to treatment on left, 5 months after application of UltraCal XS on right. UltraCal still in canals.

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